Introduction
The Ergonomic Task Chair complies with the regulations set in SS EN 1335-1 and SS EN 1335-2. It also complies with the Möbelfakta standard as well as MDR (EU) 2017/745.

Service
If you require assistance with the assembly or operation of your Ergonomic Task Chair please contact Support Design or your local dealer. Our staff will assist you in correcting the problem. Please have the serial number from the product label located under the seat before contacting Support Design or your distributor with questions regarding your unit.

Intended purpose
The Ergonomic Task Chair is intended to prevent and alleviate back problems of the user during prolonged static work and to alleviate further back problems in users who already have problems with their back by relieving the back. The device has a variety of special features, e.g. adjustable height, tilt- and swivel function, and can be equipped with accessories, e.g. backrest, armrest, footrest and foot control.

Indications
Prevents and alleviates back problems of the user during prolonged static work and alleviates further back problems in users who already have problems with their back by relieving the back.

Warnings and Contradictions

- Only use original spare parts from Support Design.
- Only use the chair for the intended purpose.
- Never use the chair on surfaces with level differences.
- Never service the chair while it is in use
- Always follow the recommended maintenance and cleaning instructions.
- Risk of shearing or squeezing fingers in moving parts.
- Do not use balance chairs if you have problems with balance.
- In case of an allergic reaction when in direct contact with the Ergonomic Task Chair, stop using the chair and consult a doctor.

Safety
The product should only be used as intended or it might cause injury to persons or product. Necessary knowledge of the product is required before use. No accessories other than those mentioned in this manual may be used. The product should only be repaired by Support Design or by a representative authorised by Support Design. Unauthorized repairs and modifications may result in loss of function, injury and void warranty.

The chair is not approved for use as a passenger seat in any means of transportation and may not be used as such.

No chemical substances listed on ECHA's candidate list of SVHC substances does occur in concentrations above 0.1% by weight.

Any serious incident that has occurred to the user in relation to The Ergonomic Task Chair should be reported to Support Design AB and the competent authority of the Member State in which the user is established

Warranty
We guarantee that your new Support Design product is free from defects in materials or workmanship. This warranty applies only to products purchased directly from Support Design or a Support Design Authorised Retailer. This warranty is valid for 3-5 years and only for the original purchaser and terminates if you transfer, sell or modify your product. Please note that our warranty does not apply to:

- Normal wear and tear
- Failure to assemble, install, or maintain the product according to Support Design's instructions and guidelines (this includes the appliance of substances which may damage the upholstery)
- Fading, discolouration or natural variations occurring in leather and fabric
- Variations occurring in surface materials (for example, colour fastness or matching grains, textures and colours across different surfaces)
- Abuse, misuse, or accident, e.g. scratches or cuts caused by the user.

Cleaning instructions
To get the most out of your chair, we ask you to read the instructions below. Please bear in mind that refraining from following these instructions may affect your warranty.

Genuine Leather
Vacuum often with a soft brush, clean the leather when needed by only using leather cleaning products. Do not expose the leather to strong sunlight or close contact with hot surfaces.

Artificial Leather
The upholstery should be cleaned regularly to maintain its surface and to prevent the build-up of dirt and bacteria. All stains, oil, grease, ink, dirt or other soiling must be washed off immediately to reduce the risk of permanent stains. Use mild soap solution/detergent or special cleaners for fabric plastics to remove stains from the surface. Use together with a damp sponge or a microfibre cloth. Most healthcare disinfectants are also fine to use, provided they are diluted according to the manufacturer's specification. Never use lacquers, abrasives, solvents, hand disinfectants, chloride, wax polish, strong cleaning agents, xylene-based solutions or acetone as these can cause immediate permanent stains on the surface of the woven plastic.

Metal Surfaces
May be cleaned using standard cleaning products. The top part of the gas column must be kept clean and dirt should be removed immediately to avoid permanent damage. If the top part of the gas spring is damaged (deep cuts, no surface scratches), it should be replaced immediately. Castors should be vacuumed and picked from dirt regularly.

Technical Specification

| | |
|---------------|--------------------------------|
| Load capacity | 130kg |
| Seat heights | 44-92cm (Depending on variant) |
| Weight | 9-17kg (Depending on variant) |
| Foot print | 50-61cm (Depending on variant) |
| Warranty | 5 Years |

Gas columns are available in 3 different heights.

Assembly Instructions
For chair that is not delivered fully assembled, please follow the steps shown in figure 1-3:
1. Insert the five casters into the base.
2. Insert the wide part of the gas column into the base.
3. Place the seat assembly on top of the gas column and press down several times to secure components.

Adjustments
Height Adjustment
Figures 1-8, shows adjustment for seat height. A correct seat height is achieved when there is an open angle between upper body and legs.

Backrest Height Adjustment
Figures 9-12, shows adjustment of the backrest height. The backrest should be adjusted to give a good lumbar support.

Seat Tilt
Figures 13-18, shows the adjustment of the seat tilt. The seat should be tilted slightly forward to achieve an open hip angle.

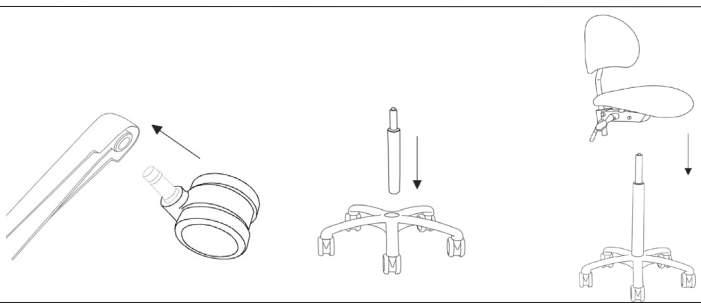
Backrest Tilt
Figures 19-22, shows the backrest tilt. The backrest should be tilted forward to give a good lumbar support.

Correct seating position
Figures 23-24, shows a correct seating position.

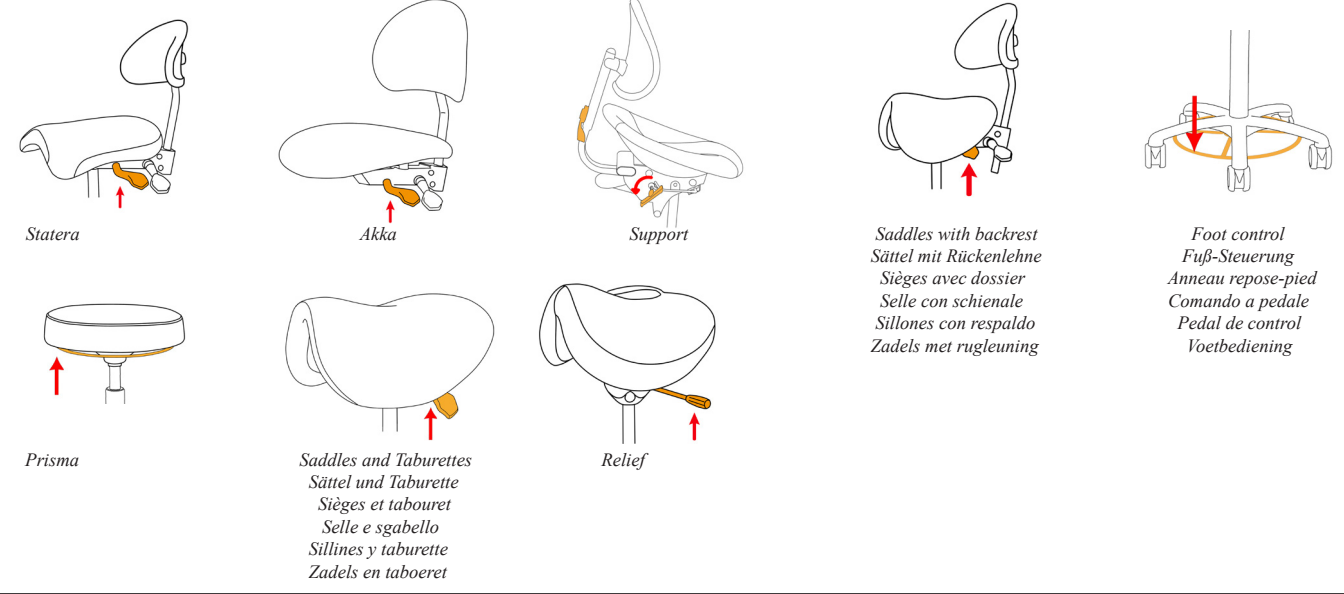
Accessories
The Ergonomic Task Chair can be equipped with a wide range of accessories, shown in figure 28.

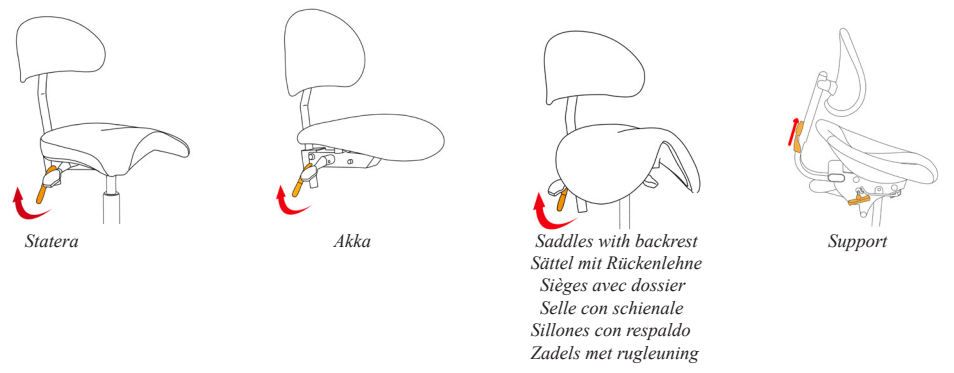
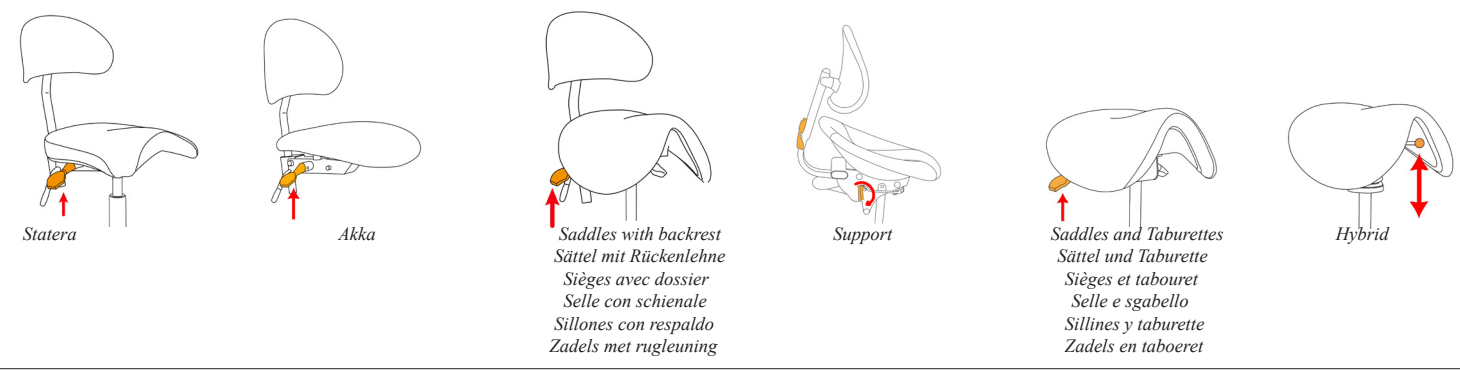
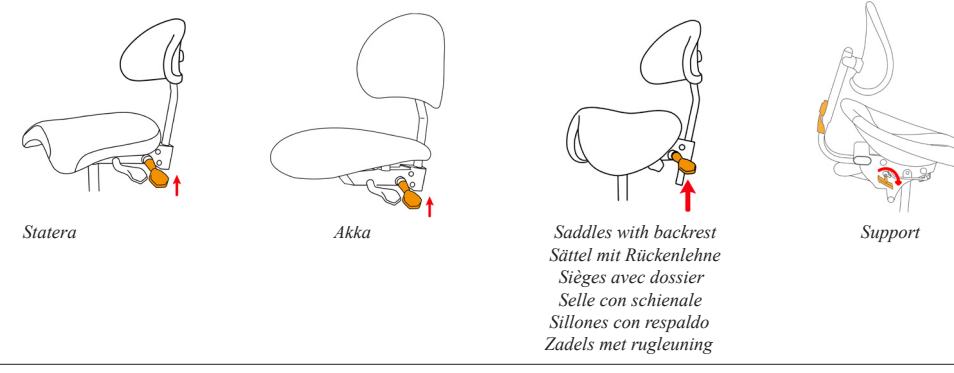
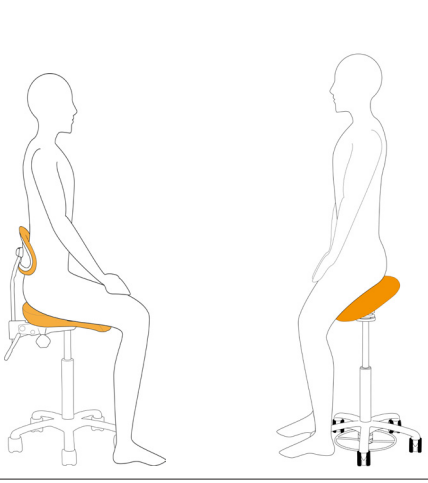
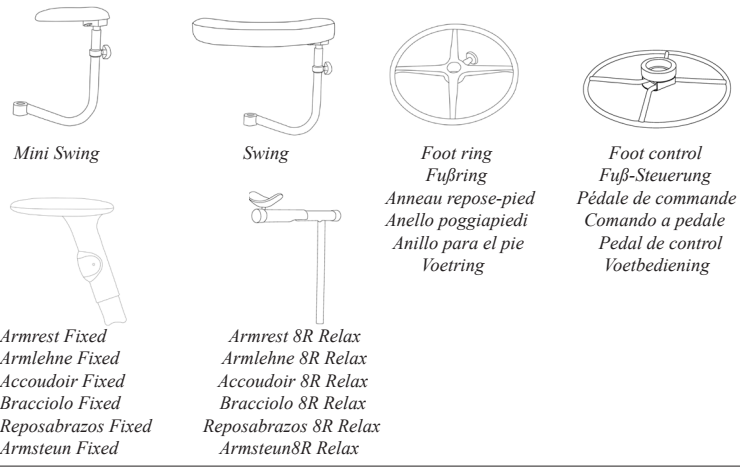
Please note: When the Ergonomic Task Chair are equipped with armrests, a base of minimum 600mm is required to ensure stability.

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